
TPIV CLAMP & BRACKET STAND ASSEMBLY INSTRUCTIONS

English

PARTS LIST		
Item	Description	Qty
1	Clamp & Bracket	1
2	Screws, 8-32 x 7/8 FH	2
3	Max Height Label	1
4	Carton	1
5	Instruction Sheet	1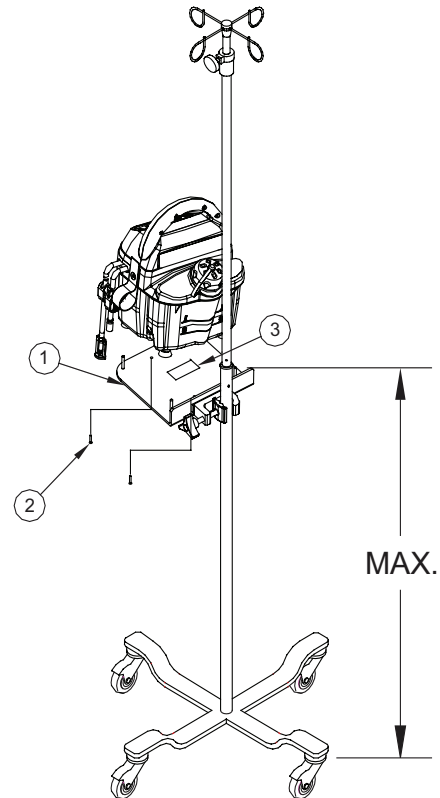

CAUTION

Do not install TPIV above maximum height (36in or 91.4cm)

PROCEDURE

1. Secure bracket to IV pole with clamp, with the post.
2. Position T/Pump® between the posts on the bracket. T/Pump® can be permanently affixed to the bracket with the included screw.

CLEANING

The TPIV stand can be cleaned with a damp cloth and mild detergent.

WARRANTY

Stryker equipment and products are warranted against defects in material and workmanship under normal use, and operation from the date of purchase, for the time periods listed below for the respective equipment and products. Except for such warranty, Stryker disclaims all other expressed and/or implied warranties including, but not limited to, the implied warranties of merchantability and of fitness for a particular purpose.

All labor performed and parts provided free of charge for a period of one (1) full year from the date of purchase, provided the equipment is returned with prior authorization prepaid to Stryker Industries.

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