
TPS2 STAND ASSEMBLY INSTRUCTIONS

English

PARTS LIST		
Item	Description	Qty
1	Steel Base	1
2	Washer	1
3	Tube	1
4	LG Basket	1
5	Mount Blocks	2
7	Washer	2
8	Cap Nut	2
9	Screw	2
10	Mount Plate	1
11	Screw	2
12	Bolt	1
PACKAGING		
NOT SHOWN	Wrench	1
	Carton	1
	Instruction Sheet	1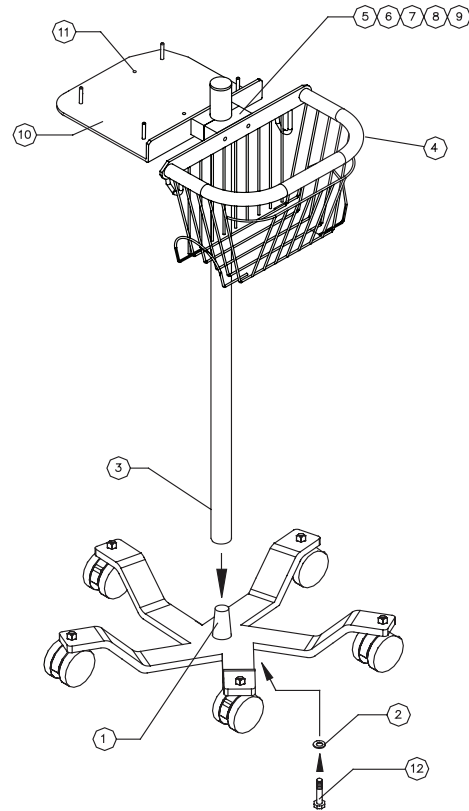

PROCEDURE

1. Press tube onto base. Secure with washer and screw.
2. Place mounting blocks around tube. Blocks should be placed over the locating pin near the top of the tube.
3. Place the screws through the mount plate and mount blocks. The posts on the mount plate must be on the top. Secure with the washer and cap nut.

Position the T/Pump® between the posts on the bracket. T/Pump® can be permanently affixed to the bracket with the included screws.

CLEANING

The TPS2 stand can be cleaned with a damp cloth and mild detergent.

WARRANTY

Stryker equipment and products are warranted against defects in material and workmanship under normal use, and operation from the date of purchase, for the time periods listed below for the respective equipment and products. Except for such warranty, Stryker disclaims all other expressed and/or implied warranties including, but not limited to, the implied warranties of merchantability and of fitness for a particular purpose.

All labor performed and parts provided free of charge for a period of one (1) full year from the date of purchase, provided the equipment is returned with prior authorization prepaid to Stryker Industries.

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Stryker European Operations B.V.