EN
2

**NGage® Salivary Stone Extractors and
NCircle® Tipless Salivary Stone Extractors**

Instructions for Use

DA
3

**NGage® spytstenekstratorer og
NCircle® spytstenekstratorer uden
spids**

Brugsanvisning

DE
4

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Bruksanvisning

NGAGE® SALIVARY STONE EXTRACTORS AND NCIRCLE® TIPLESS SALIVARY STONE EXTRACTORS

CAUTION: U.S. federal law restricts this device to sale on or by the order of a physician (or properly licensed practitioner).

DEVICE DESCRIPTION

The NGage is a 3-wire nitinol basket with a 1.7 Fr shaft diameter, and 8 mm wide basket diameter when fully expanded. This basket is compatible with endoscopes with a working channel inner diameter of 0.8 mm or greater.

The NCircle is a 4-wire nitinol basket with a 1.5 Fr shaft diameter, and 1 cm wide basket diameter when fully expanded. This basket is compatible with endoscopes with a working channel inner diameter of 0.65 mm or greater.

INTENDED USE

The NGage and NCircle Salivary Stone Extractors are intended for manipulation, entrapment and extraction of calculi and other foreign bodies in the salivary ducts, and to minimize migration of salivary stones during interventional procedures.

CONTRAINDICATIONS

- Sialadenitis
- Acute Glandular Infection

WARNINGS

This device is not intended to be used concomitantly with an electrocautery power source. This device removes salivary stones by mechanical force alone, and should not be attached to any power source.

PRECAUTIONS

- This product is intended for use by physicians trained and experienced in sialendoscopy. Standard sialendoscopic techniques should be employed.
- Assess calculi and other foreign bodies prior to instrument deployment to ensure that the object is not too large to be removed through the anatomy. (Factors predicting successful stone retrieval with sialendoscopy-assisted basket retrieval include stone size (less than 3 or 4 mm for parotid and submandibular stones, respectively), shape (long and narrow or round), orientation (largest diameter parallel to duct), distal location of the stone in the main salivary duct, and if calculus is free-floating after ductal irrigation. Note that each patient's anatomy will vary.¹⁻³)
- If resistance is encountered while attempting to remove calculi or other foreign bodies, release the object. Excessive force could damage the device.
- If the basket does not readily release the captured object, further manipulation may be necessary.
- During device withdrawal, take care to keep the shaft as straight as possible. Withdrawing the device at an excessive angle or curvature may damage the shaft.

POTENTIAL ADVERSE EVENTS

The following complications are possible with the use of the stone extraction devices:

- Pain
- Bleeding
- Infection
- Non-Retrieval of Stone
- Tissue Perforation
- Tissue Avulsion
- Entrapment of basket around stone requiring surgical removal.

INSTRUCTIONS FOR USE

1. Remove the stone extractor from its outer package and move the thumb lever on the handle to the "Close" position to close the basket.
2. Insert the basket through the instrument channel of the endoscope until the tip is visualized in the salivary duct.
3. Under direct visualization, locate a stone or foreign body to remove. Slide the thumb lever on the handle to the "Open" position to deploy the basket near the object, and maneuver the device until the object is within the basket.
4. Once correctly positioned, draw the basket down around the stone or foreign body by sliding the thumb lever towards the "Close" position until resistance is felt and the basket is secure around the stone.

NOTE: Do not use excessive force on the handle. Doing so may result in damage to the device.

5. Maintaining position of the stone or foreign body within the basket, retract the endoscope and basket as a unit from the salivary duct to facilitate immediate removal of the object.

6. Advance the endoscope back into the salivary duct to identify any remaining stone or foreign body fragments to be removed. Repeat steps 2-5 as needed for additional stone fragments.

HOW SUPPLIED

Supplied sterile by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened or undamaged. Do not use product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid exposure to light. Upon removal from the package, inspect the product to ensure no damage has occurred.

REFERENCES

These instructions for use are based on experience from physicians and (or) their published literature. Refer to your local Cook sales representative for information on available literature.

1. Marchal F, Dulguerov P. Sialolithiasis management. *Archives of Otolaryngology—Head & Neck Surgery*. 2003;129(9):951. doi:10.1001/archotol.129.9.951.
2. Walvekar RR, Carrau RL, Schaitkin B. Endoscopic sialolith removal: Orientation and shape as predictors of success. *American Journal of Otolaryngology*. 2009;30(3):153–156. doi:10.1016/j.amjoto.2008.03.007.
3. Zenk J, Koch M, Klintworth N, et al. Sialendoscopy in the diagnosis and treatment of Sialolithiasis: a study on more than 1000 patients. *Otolaryngology—Head and Neck Surgery*. 2012;147(5):858–863. doi:10.1177/0194599812452837.

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© COOK 2019
Bloomington, IN 47404 U.S.A.
Cook Medical Europe Ltd