


A transformative approach to traditional *butterfly graft* tympanic membrane repair^{1,2}

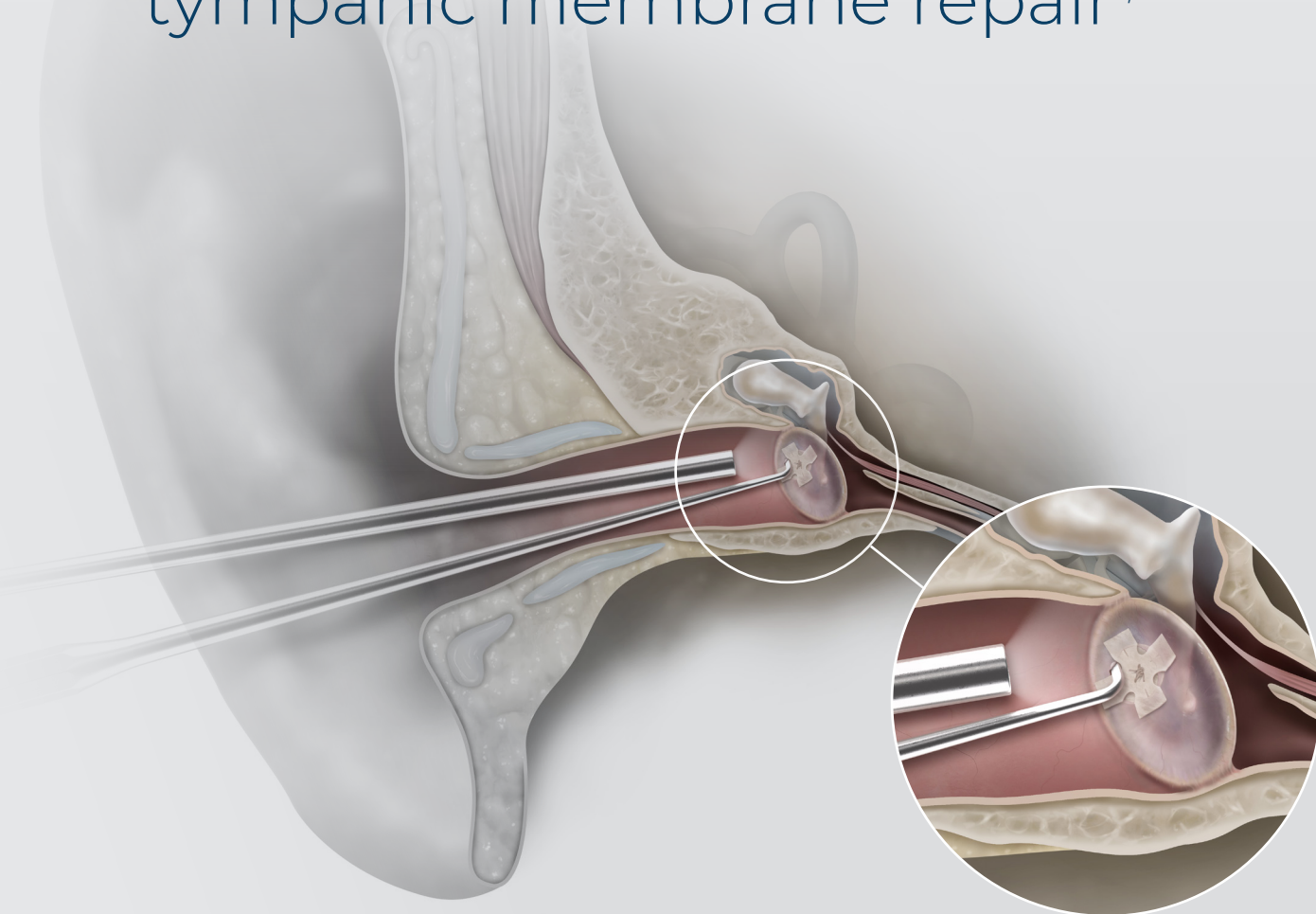

Biodesign[®] OTOLOGIC BUTTERFLY GRAFT

Reliable closure

Biodesign material remodels into natural host tissue with an overall success rate of 91% across published literature³⁻¹¹ with no statistically significant difference in audiometric results when compared to temporalis fascia.^{3,12}

Excellent handling

Biodesign material is easy to manipulate, allowing for improved surgical precision during graft placement.³

Time saving

The Biodesign Otologic Butterfly Graft is a new, off-the-shelf option for a minimally invasive procedural approach. It reduces the need to harvest autologous tissue, significantly decreasing intraoperative time.³

Biodesign[®]

OTOLOGIC BUTTERFLY GRAFT

Tips to help get the best possible results:

- Ensure adequate blood supply.
- Size the graft to allow some tissue overlap.
- Place the graft dry or hydrate it for less than one minute before placement.

Available product sizes

Shown at actual size.



Biodesign Otolgic Butterfly Graft


Order Number	Reference Part Number	Size cm	Nominal Thickness mm
G60285	ENT-OTO-BFLY-0.4-0.6	0.4, 0.6	0.25

Some products or part numbers may not be available in all markets. Contact your local C2Dx representative or Customer Support for details.

References

1. Internal white paper; "Evaluation of industry opinion on endoscopic transcanal TM repair as a conservative and minimally invasive procedure". Available on request.
2. https://www.accessdata.fda.gov/cdrh_docs/pdf23/K232646.pdf; Published K232646 510k summary describing device usability testing data in benchtop and animal models following an endoscopic transcanal approach.
3. D'Eredità R. Porcine small intestinal submucosa (SIS) myringoplasty in children: a randomized controlled study. *Int J Pediatr Otorhinolaryngol.* 2015;79(7):1085-1089.
4. Cass ND, Hebbe AL, Meier MR, et al. Pediatric primary tympanoplasty outcomes with autologous and non-autologous grafts. *Otol Neurotol.* 2022;43(1):94-100.
5. Chen CK, Hsieh LC. Clinical outcome of exclusive endoscopic tympanoplasty with porcine small intestine submucosa in 72 patients. *Clin Otolaryngol.* 2020;45(6):938-943.
6. Barron C, Lukens J, Niermeyer W, et al. Investigation of novel grafts in use for pediatric tympanoplasty. *Ann Otol Rhinol Laryngol.* 2019;128(12):1111-1115.
7. Redaelli De Zinis LO, Berlucchi M, Nassif N. Double-handed endoscopic myringoplasty with a holding system in children: preliminary observations. *Int J Pediatr Otorhinolaryngol.* 2017;96:127-130.
8. James AL. Endoscope or microscope-guided pediatric tympanoplasty? Comparison of grafting technique and outcome. *Laryngoscope* 2017;127(11):2659-2664.
9. Ranguis SC, Leonard CG, James AL. Prospective comparison of pediatric endoscopic lateral graft and interlay tympanoplasty. *Otol Neurotol.* 2021;42(6):867-875.
10. Wang N, Isaacson G. Collagen matrix as a replacement for Gelfilm for post-tympanostomy tube myringoplasty. *Int J Pediatr Otorhinolaryngol.* 2020;135:110136.
11. Yawn RJ, Dedmon MM, O'Connell BP, et al. Tympanic membrane perforation repair using porcine small intestinal submucosal grafting. *Otol Neurotol.* 2018;39(5):e332-e335.
12. Dontu P, Shaigany K, Eisenman DJ. Anatomic and audiometric outcomes of porcine intestinal submucosa compared to autologous fascia for tympanic membrane repair. Poster presented at: Combined Otolaryngology Spring Meetings, COSM 2022; April 27 – May 1, 2022; Dallas, TX.

© indicates U.S. trademark registration. All trademarks and/or images are the property of their respective owners or holders.

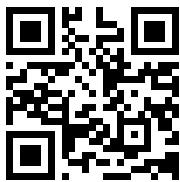

For more detailed product information, including the Instructions for Use, which contains the indication statement, contraindications, precautions, and potential complications, visit www.C2Dx.com.